## Taschenbeatmungsmaske

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### Instruction manual

#### 1 Use & function

Our pocket ventilation mask is used to ventilate adults in an emergency, with the aim of maintaining the supply of vital oxygen to the organism. The integrated valve protects the first aider during mask-to-mouth ventilation by expelling the patient's used exhaled air to the side. The additional built-in fleece in the valve serves as a mechanical barrier against liquids. The inhibition threshold through mouth contact is reduced and infection through direct contact with pathogenic fluids and particles from the patient is minimized. Prerequisite for use: The first responder should have training or experience in the use of respiratory aids.

**⚠ Note:** In case of a simultaneous cardiac arrest, please observe the current guidelines for cardiopulmonary resuscitation (CPR): 30 × chest compressions followed by hyperextension of the head and 2 × respiration.

#### 2 Storage

For hygienic reasons, the mask should always be kept in the box.

#### 3 Technical characteristics

##### Temperature ranges:

Application: -18° C +40° C

Storage: -30° C +50° C

##### Materials:

Mask: polyvinyl chloride (PVC)  
Valve: nonwoven

##### Dead space volume:

170ml

##### Respiratory- and inhalation resistance:

Inspiratory resistance: 3,5 cmH<sub>2</sub>O

Expiratory resistance: 2,5 cmH<sub>2</sub>O

##### Size of oxygen compound:

6 mm

##### Dimensions :

Storage box: approx. 13 × 10,5 cm (L × W)

Loop: approx. 15 cm (L)

Mask, folded: approx. 12 × 10 × 2cm (L × W × H)

Mask, opened with valve: approx. 12 × 10 × 6cm (L × W × H)

**⚠ Note:** Mask does not provide 100% protection against infectious diseases

**⚠ Note:** Do not use the respiratory mask if the packaging is damaged or has already been used.

**⚠ Caution:** Not suitable for toddlers and neonates. Here are other measures adapted to the patient.

PVC-components may contain phthalates (see marking). Take special care with: newborns, pre-pubertal children, breastfeeding and pregnant women.

#### 4 Operation & Application

Open the hardcase. Remove the mask and the connector. Control the airways for foreign bodies. If present, remove them.

**1** Connect the connector to the mask.

**2** Press the mask into shape.

**3** Place the mask with the tapered side pointing forwards over the mouth and nose on the patient's face. Be sure to fix the mouth in a slightly open position.

**4** Fix the mask with the rubber band behind the head. Stretch the patient's head.

**5** Fix the mask firmly on the face with C-grip and carry out the respiratory donation. Cover the head of the patient.

**6** Check if the rib cage rises. If this is not the case, check the fit of the mask and correct it if necessary.

Continue with the chest compressions until the patient's breathing regains its independence.

#### 5 Explanations

Zeichen	Bedeutung
	CE marking
	Medical device
	Latex-free
	Order number
	Protect from heat and light
	Manufacturer
	Expiry date
	Single-use only
	Follow the instruction manual
	Batch code
	Date of manufacture
	Contains phthalates
	Recyclable
	Not sterile
	Importer
	Do not use if package is damaged
	Protect from moisture

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© HERZmed GmbH